


Introducing the FLS practice base

The FLS-PB09 fits into our MITS- TRLCD05 trainer and provides the same working platform as the FLS trainer

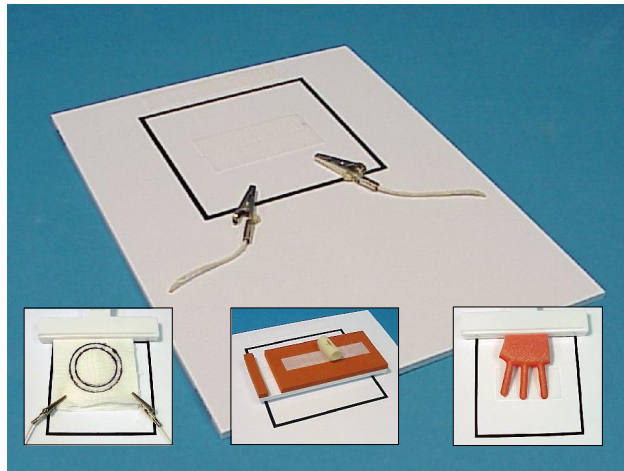



FLS-PB09

Features:

- Practice base mounts in trainer with Velcro[®] fasteners
- Black line defines the working area
- Velcro strips for “Jumbo Clip”, peg game and penrose holder
- Alligator clips for holding gauze pad

Installs in seconds

1. Move the SimScope to the center port of trainer.
2. Clean and dry the trainer interior base surface.
3. (On backside) remove backing on hook side of Velcro[®]
4. Insert FLS-PB09 into trainer - align with inside front of trainer

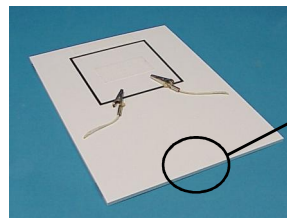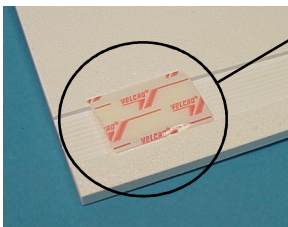

5. Press on practice board to adhere Velcro[®] adhesive to trainer base.

Residents may practice on the MITS- TRLCD05 trainer to acquire the skills incorporated into the FLS modules. However, FLS certification testing must be done at one of the official FLS Test Centers by an FLS proctor using the official FLS Laparoscopic Trainer Box. For more information please visit <http://www.flsprogram.org>.