



**SIGMOIDOSCOPIC
EXAMINATION SIMULATOR
LF00984U
INSTRUCTION MANUAL**



Life/form® Products by NASCO



About the Simulator...

Proven effective, the **Life/form**[®] Sigmoidoscopic Examination Simulator is an excellent teaching aid for training health care professionals in all aspects of the flexible sigmoidoscopic examination. Originally produced exclusively for ACMI, this simulator has been used for physician training nationwide.

List of Components

- Simulator on Oak Base
- Large Syringe
- Instruction Manual
- Carrying/Storage Case

General Instructions for Use

Treat this simulator as if it were an actual patient. Excessive pressure, failure to properly lubricate, etc., can cause damage. The prolonged life and use of this simulator depends on proper handling.

This simulator offers all the challenges encountered with real patients; the value of examining the entire length of the large intestine has been established, and this unit provides all the practice necessary for confident transfer of applied techniques.

The first step in using this unit is to lubricate it, but don't use just any lubricant; the recommended lubricant is water. Use the large syringe supplied and squeeze a generous supply into the simulator.

Once all instrumentation has been lubricated, proceed as per the instructor's advice. Proper instruction and practice will develop the necessary hand/eye coordination to allow passage of the sigmoidoscope through the entire length of the intestine.

As the scope follows the contours of the large intestine, the student must negotiate the sigmoid colon, splenic flexure, and hepatic flexure; thereafter, viewing of the ascending colon and cecum is possible.

As the scope is withdrawn, the student can identify benign sessile, pedunculated, and hyperplastic polyps, as well as cancerous lesions. There are a total of 12 pathological conditions to diagnose. All manipulative and photographic techniques can also be practiced.

Care of the Simulator

Normal surface soil can be removed with mild soap and lukewarm water. Use REN Cleaner (W09919U) to remove stubborn stains from the simulator. Simply spray soiled area and wipe clean with a soft cloth or paper towel.

Cautions

- DO NOT mark on the simulator in any way.
- DO NOT allow any newsprint to come into contact with the simulator. Ink, of any type, will cause indelible stains.
- DO NOT use a substitute lubricant; water works best.

Supplies/Replacement Parts for the Life/form[®] Sigmoidoscopic Simulator:

W09919U REN Cleaner

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01037U Hemodialysis Arm
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01121U Advanced IV Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF03965U Deluxe "Plus" **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin
LF06001U **CPR Prompti**™ Adult/Child Manikin
LF06012U **CPR Prompti**™ Infant Manikin
LF06200U **CPR Prompti**™ Keychain Rescue Aid
LF06204U **CPR Prompti**™ Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNasco.com • E-mail: lifeform@eNasco.com